

Aliaxin® I'M PERFECT



PERSPECTIVE OF BEAUTY OVER TIME.
Personal portrait.

EVERYONE IS A MASTERPIECE.



Caring Innovation

IBSA DERMA PILLARS

IBSA Derma offers a full range of products and brands such as Viscoderm®, Profhilo® and Aliaxin® based on the Hydrolift® Action concept.



Hydrolift® Action is an innovative approach aimed at counteracting the physiological reduction of hyaluronic acid in the skin, restoring hydration, elasticity and skin tone.

Hydrolift® Action is an expression of the synergy of selected hyaluronic acid, produced with the patented IBSA technology which, used in combination, creates optimal conditions for preventing and fighting the aging process.

HYALURONIC ACID AT THE HEART OF OUR TECHNOLOGY

At the core of all the technologies utilized in the production of IBSA Derma products is the aim to preserve as much as possible the natural characteristics of hyaluronic acid while at the same time enhancing them.

In fact, all the different formulas, including cross-linked ones, are based on the use of hyaluronic acid chains with different molecular weights that perform different actions in the surrounding tissue.

This gives us the opportunity to offer a high level of cohesivity that facilitates integration into the tissue itself.

HA SHYALT ULTRAPURE

IBSA Derma hyaluronic acid “ultrapure” is characterized by its high degree of purity, produced through a patented biofermentation process that creates a very high quality ingredient.



AGING & TAILORING AESTHETIC MEDICINE

Baby Boomers and Gen ZS currently comprise the largest segments on aesthetic medicine markets.

Cumulative aging of the face layers over time leads to *changes in all face structural tissues; the morphology in terms of shape, proportions and topography*¹

These variations can significantly influence **filler based rejuvenation approaches**, making **personalized approach** essentials to achieve optimal outcome²

1) An approach to structural facial rejuvenation with fillers in women R. Fitzgerald, MD, J. Carqueville, MD P.T. Yang, MD, FRCSC.

2) Plast Reconstr Surg Glob Open 2020;8:e2763; doi: 10.1097/GOX.0000000000002763; Published online 15 October 2020.

Aliaxin® SR



HYALURONIC ACID: MOLECULAR WEIGHTS

500 kDa
1000 kDa
2000 kDa

TOTAL HA CONTENT

25 mg/ml

RHEOLOGICAL VALUES³

Flowability =
Tan delta (δ): 0.53
Elastic Modulus =
G' (Pa): 39

NEEDLES

30G x 13 mm
27G x 19 mm

INDICATIONS

Forehead
Temples (superficial)
Malar area

GPS SCALE



COHESIVITY



FLOW



LIFT



SCULPT

Aliaxin® GP



HYALURONIC ACID: MOLECULAR WEIGHTS

1000 kDa
2000 kDa

TOTAL HA CONTENT

25 mg/ml

RHEOLOGICAL VALUES³

Flowability =
Tan delta (δ): 0.23

Elastic Modulus =
G' (Pa): 95

NEEDLES

27G x 13 mm
27G x 19 mm


INDICATIONS

Glabella
Nasolabial folds
Marionette lines
Tear trough

GPS SCALE



COHESIVITY

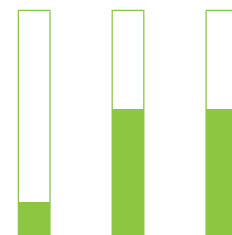

FLOW

LIFT

SCULPT

GENTILIZE PROTOCOL

“Specialists have to look at patient face with different eyes, enhancing their skills in order to identify what procedures create that awe, that delightful feeling of wonder that everyone aspires to.”




Editta Buttura Da Prato, MD




GENERAL ASSESSMENT

- Not requiring any volumization effect since the volumes are still present
- Shape of major fat compound are well represented
- Thin skin and slightly subderma atrophy
- Missing is a face contour

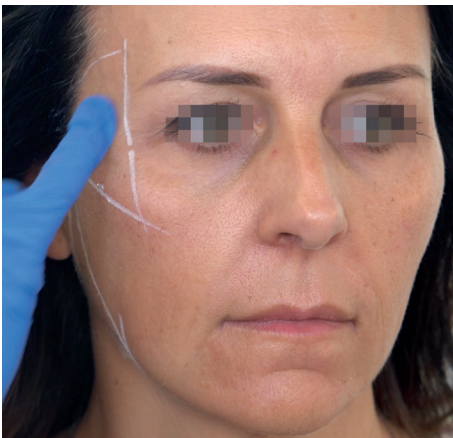

PATIENT ASSESSMENT

- 50 years old pre-treated
- Not missing volumes
- Requires working on face oval
- Discrepancy between temporal fossa and zygomatic projection
- Posterior pulling to lighten the weight of sagging tissues on the mandibular ligament

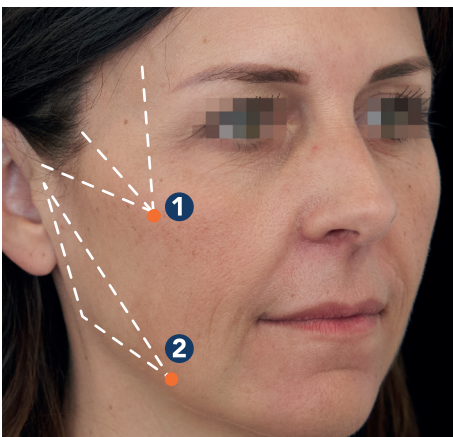

EXECUTION

- ① Point of maximum zygomatic projection, working on temporal and zygomatic area
fanning technique – 50mm cannula – 25G
fanning technique – 50mm cannula – 25G
- ② Masseter anterior border with entry point in the mandibular line;
fanning technique – 50mm cannula – 25G



Aliaxin® SR for gentle stretching.

BEFORE

AFTER



TREATMENT	Mandibular angle
	1 ml per side
TREATMENT	Cheekbones
	1 ml per side

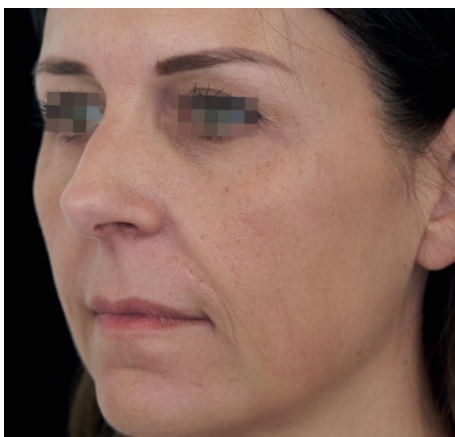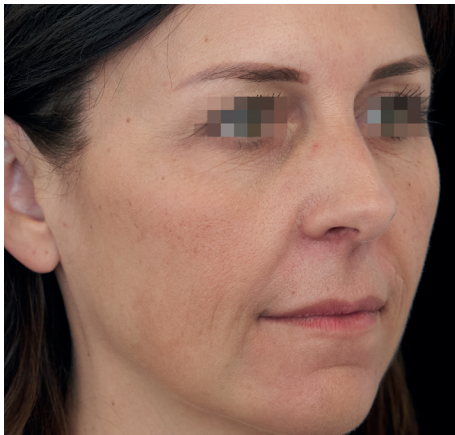

Courtesy of Dr. Editta Buttura Da Prato



HARMONIZE PROTOCOL

“Looking at Aesthetic Medicine from an international perspective, there are both common trends and substantial differences, the latter being mainly related to local aesthetic preferences and culture.”



Beatriz Molina, MD




GENERAL ASSESSMENT

- Volumes are present
- Missing shape or projection for a slight initial fat pad atrophy
- Not need for strong volumization but for gentle enhancement
- Visible initial marionettes lines
- Visible nasolabial fold wrinkles



PATIENT ASSESSMENT

- 58 years old pre-treated
- Thick skin and volumes present
- Missing lateral projection for initial fat pad atrophy
- Initial nasolabial fold

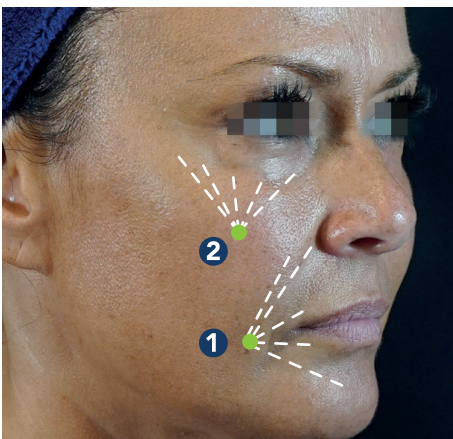

EXECUTION

① 1 cm from the lateral corner of the mouth

Nasolabial: Linear retrograde 2/3 steps (similar fanning) - 38mm cannula - 25G - subdermal, superficial fat pad

Marionettes: Linear retrograde 2/3 steps (similar fanning) - 38mm cannula - 25G - subdermal, superficial fat pad deep fat pad under the ligament under the muscle

② 1.5 cm below the point of maximum zygomatic projection

fanning technique - 38mm cannula - 25G - deep fat pad under ligament under muscle



Aliaxin® GP for minor enhancement and natural results.

BEFORE

AFTER



TREATMENT	Cheekbones
	1 ml per side
TREATMENT	Corner of the mouth (nasolabial + marionette lines)
	0.8 ml per side

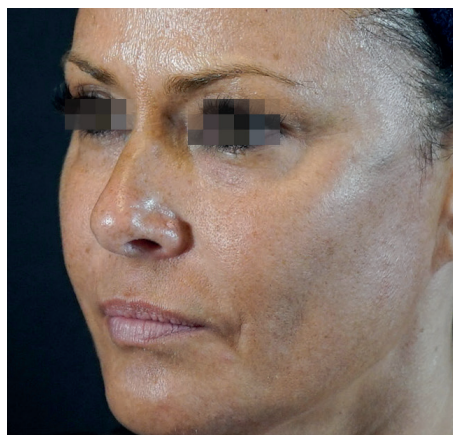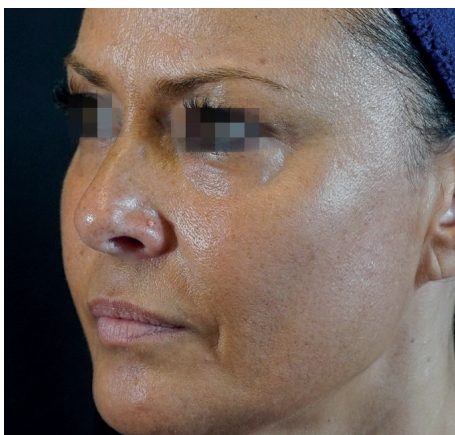

Courtesy of Dr. Beatriz Molina



IBSA Farmaceutici Italia S.r.l
Via Martiri di Cefalonia 2 - 26900 Lodi - Italy

🌐 ibsaderma.com

@ info@ibsaderma.com

📘 [ibsaderma](#)

📷 [ibsaderma](#)



PAEEZAN
TEJARAT MEHR

📍 Paezan Tejarat Mehr
Unit 1, 2nd Floor, 60 East
Maryam St, Elahiye Ave, Tehran,
19649-66335, Iran

☎ +98 21 220 44 991-3
+98 21 262 91 556

🌐 www.paezanco.com



📷 [ibsadermairan](#)

Aliaxin[®]

IS A CLASS III MEDICAL DEVICE



Rose Pharma S. A. Via San Gottardo, 10
6900 Lugano, Switzerland

CE 0373



Material intended for medical practitioner's use only

ALI-BRM-001-2023-E

7500002993